

### 3. Injection

Administer medication in accordance with established protocol. For user convenience, the needle "bevel-up" position is oriented to the safety cover.



### 4. Activation

After injection, immediately activate safety cover as follows:

- Using the same hand holding the syringe, center thumb or forefinger on the finger pad area of the safety cover.
- Push the cover forward toward the needle until you hear and/or feel it lock. Visually confirm that the needle tip is covered.



*Discard after single use in an approved sharps container.*

### Don't Forget...

Aseptic technique and proper skin preparation of the site are essential. Follow your institution's policy for safe disposal of all medical waste.

#### BD Eclipse™ Needle for Luer Lock Syringes - 100/Box - 1200/Case

Catalog #	Needle Size	Catalog #	Needle Size
305761	25G x 1"	305765	21G x 1½"
305767	25G x 1½"	305766	18G x 1½"
305762	23G x 1"	305759	25G x ⅝"
305768	22G x 1"	305758	27G x ½"
305763	22G x 1½"	305757	30G x ½"
305764	21G x 1"		

#### BD Eclipse™ Needle with Luer Lock Syringe 1mL BD Luer-Lok™ Syringe with Detachable Needle - 50/Box - 300 Case

305778	30G x ½"	305780	25G x ⅝"
305789	27G x ½"		

#### 3mL BD Luer-Lok™ Syringe with Detachable Needle - 50/Box - 300 Case

305781	25G x ⅝"	305783	22G x 1½"
305787	25G x 1"	305779	21G x 1"
305782	23G x 1"	305784	21G x 1½"
305788	22G x 1"		

#### 5mL BD Luer-Lok™ Syringe with Detachable Needle - 50/Box - 300 Case

305785	22G x 1 ½"		
--------	------------	--	--

#### 10mL BD Luer-Lok™ Syringe with Detachable Needle - 50/Box - 300 Case

305786	22G x 1 ½"		
--------	------------	--	--

#### For Luer Slip Syringes - BD Eclipse™ Needle with SmartSlip™ Technology 100/Box-1200 Case

305760	25G x ⅝"	305771	30G x ½"
305770	27G x ½"		

#### 1mL Luer Slip Syringe with Detachable Needle - 50/Box - 300 Case

305775	30G x ½"	305777	25G x ½"
305776	27G x ½"		

**REMEMBER:** Safety-Engineered Devices are required in your Custom and Standard Procedure Trays.

For more information call  
**1.888.237.2762**

BD, BD Logo, and all other trademarks are property of Becton, Dickinson and Company. © 2013 BD.  
BD 2/13-MSS0078

 **BD**  
BD Medical  
1 Becton Drive  
Franklin Lakes, NJ 07417  
www.bd.com




 **BD** Helping all people live healthy lives

## BD Eclipse™ Needle

# How to Use the BD Eclipse™ Needle for Luer Lock Syringes

### 1. Attachment

- Firmly attach needle onto a Luer Lock syringe with a push and clockwise twist.
- Pull back on the safety cover toward the syringe and away from the needle.
- Grasp the syringe with one hand and with the other hand pull the clear needle shield straight off.

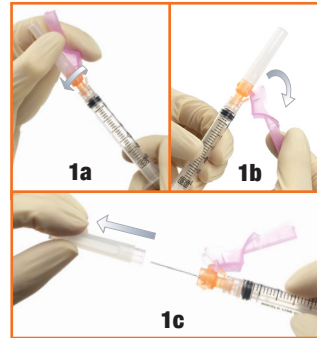

### 2. Aspiration

Draw up medication as per your usual technique.



**NOTE:** If changing needles, activate safety cover prior to removing the needle from the syringe.

Also available:

## BD Eclipse™ Needle with SmartSlip™ Technology

Expanding Safety for Luer Slip Connections




**Specialized Clip**  
Helps ensure a secure connection to Luer Slip Syringes

SmartSlip™ Technology, available on select small gauge needle sizes, is designed to help ensure a secure connection to luer slip syringes. For detailed attachment instructions for luer slip syringes, see BD Eclipse™ SmartSlip™ Needle Usage Guidelines.

**NOTE:** The BD Eclipse™ SmartSlip™ Needle is not for use with glass syringes.

## The BD Eclipse™ Needle offers a wide range of choices for luer lock and luer slip applications.

### Prior To Activation

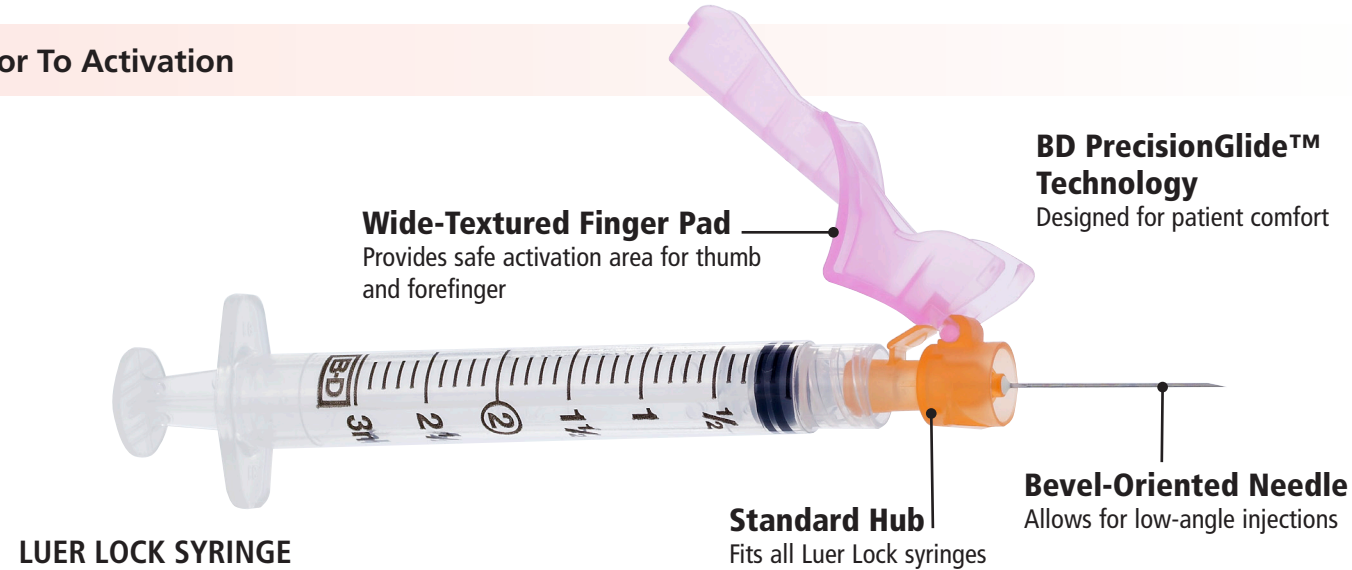

**Wide-Textured Finger Pad**  
Provides safe activation area for thumb and forefinger

**BD PrecisionGlide™ Technology**  
Designed for patient comfort

**Standard Hub**  
Fits all Luer Lock syringes

**Bevel-Oriented Needle**  
Allows for low-angle injections

**LUER LOCK SYRINGE**

### After Activation



**Needle Remains Locked Inside Activated Cover**  
Helps to eliminate exposure to needle after activation

## Backed By BD's Reputation For Safety and Quality

### BD PrecisionGlide™ Needle Technology

- Designed for patient comfort
- Leading brand of hypodermic needle technology in the U.S.

### Safe, One-Handed Activation

- Single finger stroke activation: no hard surface required
- Textured finger pad keeps fingers behind the needle
- Requires minimal change in injection technique

### Designed For Safety

- Safety cover is present between clinician and needle
- Audible click indicates safety activation
- Needle remains locked inside activated cover
- The BD Eclipse™ Family offers one safety technology for both your injection and blood collection needs

### BD SmartSlip™ Technology Options

- Available on select small gauge needle sizes
- Designed for use with luer slip syringes